

KIMTECH™


Kimtech™ A5 Sterile Integrated Hood and Mask

Donning Technique



1

Select your integrated hood and mask; inspect package for vacuum seal; tear along notches toward the product to open.

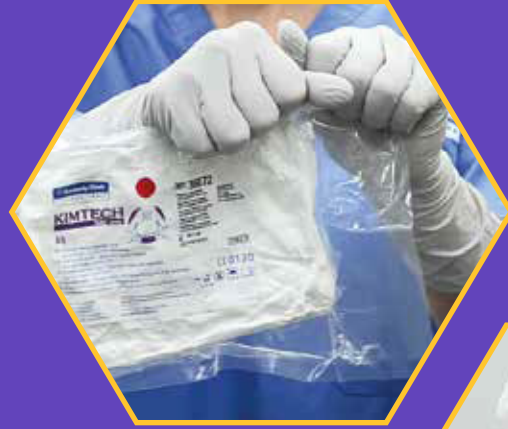

2

Pull out integrated hood and unfold to locate the blue signal indicator line.



3

Centre your chin just above the blue line then roll over your forehead.



4

Grab ties with both hands then extend arms out in a circular motion to position under arms.



5

Pull ties across chest and secure as needed.



6

Adjust front flaps to comfortable position, reach up underneath the bib of the hood, touching sterile to sterile, and pinch mask wire over the bridge of nose to ensure a secure fit.



Visit our website at www.kimtech.eu
Technical help: kimtech@kcc.com

©/™ Trademarks of Kimberly-Clark Worldwide, Inc. ©2020 KCWW Publication code ID 5372.01 EN 06.20

Distributed by Fisher Scientific. Contact us today:

Austria: fishersci.at Belgium: fishersci.be Denmark: fishersci.dk
Germany: fishersci.de Ireland: fishersci.ie Italy: fishersci.it
Finland: fishersci.fi France: fishersci.fr Netherlands: fishersci.nl
Norway: fishersci.no Portugal: fishersci.pt Spain: fishersci.es
Sweden: fishersci.se Switzerland: fishersci.ch UK: fishersci.co.uk

© 2022 Thermo Fisher Scientific Inc. All rights reserved.
Trademarks used are owned as indicated at fishersci.com/trademarks.

 **fisher scientific**
part of Thermo Fisher Scientific